

BLOOD TRANSFUSION SET

DEVICE INTENDED USE:

A blood transfusion set (BT set) is a sterile device used to deliver blood or blood products from a container into a patient's vein.

INSTRUCTIONS FOR USE:

- Remove the outer Cap of the container and disinfect the top.
- Firmly close the regulator
- Remove the spike cover and insert the spike at the top of the container to its full length, turn the container upside down, and hang it at a suitable height.
- Squeeze the drip chamber until it is filled up to the mark.
- This prevents air bubbles from entering the fluid path.
- Remove the protective cover of the needle. Gently release the regulator.

In order to let the solution flow, keep the needle above the container level.

Gradually Lower the needle so that no air bubbles are left tube.

Let a few drops of solutions pass through.

- Close the regulator.
- Perform vein puncture and release the regulator to achieve the desired flow rate.
- Use Only For Blood bags.

CAUTION :

- This is for single use discard after use.
- Do not store in direct sunlight at extreme temperatures or in high humidity
- Do not use if the package is open or damaged and the protective cap is detached

